CERUSTOP Ohrenöl

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INSTRUCTIONS FOR USE

CERUSTOP Ear Oil

Medical device for cleaning and care of the external auditory canals

COMPOSITION

Caprylic/Capric Triglyceride, Liquid Paraffin, Isostearyl Isostearate, Sweet Almond Oil, PEG-40 Sorbitan Peroleate, BHA/BHT, Vitamin E

PROPERTIES

CERUSTOP Ear Oil is used for the physical treatment of the external auditory canals. Hardened earwax affects the hearing and can cause inflammation of the external auditory canal. It should therefore be removed regularly. CERUSTOP Ear Oil dissolves hardened earwax. The oils contained in CERUSTOP Ear Oil penetrate into the earwax making it more soluble and therefore easily removable. Furthermore, CERUSTOP Ear Oil serves to remove dirt and dead skin from the ear and the external auditory canal.

CERUSTOP Ear Oil is also particularly suitable for users of hearing aids.

USE

To remove earwax, dirt and dead skin attached to the outer ear place a few drops of lukewarm CERUSTOP Ear Oil on a piece of cotton wool and wipe out the ear several times. Afterwards wipe clean with a damp cloth and then dry.

To clean the external auditory canal drop about half a pipette of lukewarm CERUSTOP Ear Oil into the canal keeping the head inclined at an angle. Close the ear with a small piece of cotton wool to prevent CERUSTOP Ear Oil from dripping out.

The cotton wool should be applied loosely and never pressed into the auditory canal!

After approximately 10 minutes, preferably lying down, rinse the ear gently with warm water if possible with a hand held shower. In cases of extreme hardness it is recommended to repeat the procedure.

CERUSTOP Ear Oil can also be used by the physician before rinsing the ear.

IMPORTANT INFORMATION

CERUSTOP Ear Oil should not be used if the auditory canals are inflamed or when the eardrum is perforated (hole in the eardrum).

CERUSTOP Ear Oil is not medicine! In case of earache or extremely hardened plugs of earwax medical advice should be sought.

The bottle should always be kept closed and out of the reach of children!

Do not store below 8 °C and not above 25 °C.

After first opening use no longer than 8 weeks.

After showering the use of NORMISON Ear Drops is recommended to quickly dry and re-establish the natural acid layer of the skin of the external auditory canal.

WARNING

Do not insert any objects (e.g. cotton buds) into the auditory canal!

Do not swallow CERUSTOP Ear Oil or insert it into the eyes!

PACKAGE SIZE

10 ml bottle with dropper pipette.

MANUFACTURER: MoNo chem-pharm Produkte GmbH, 1200 Vienna - Austria

DISTRIBUTION AUSTRIA: SIGMAPHARM Arzneimittel GmbH & Co KG, 1201 Vienna - Austria

DISTRIBUTION GERMANY: bene-Arzneimittel GmbH, 81479 Munich - Germany

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VERTRIEB DEUTSCHLAND: bene-Arzneimittel GmbH, D-81479 München