# NORMISON Ohrentropfen

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## INSTRUCTIONS FOR USE

# NORMISON Ear Drops

**Medical device for cleaning and care of the external auditory canals  
after bathing, showering, swimming and diving**

### COMPOSITION

Isopropyl alcohol, Water, Acetic acid, Dexpanthenol

### PROPERTIES

NORMISON Ear Drops are used for the physical cleaning and care of the external auditory canals after bathing, showering, swimming and diving.

The special composition of NORMISON Ear Drops makes it possible that the skin surface in the external auditory canal dries quickly.

The acidic component in NORMISON Ear Drops creates a slightly acidic environment matching the natural pH value (acidity) of the skin. Thus, NORMISON Ear Drops help to re-establish the natural acid coating of the skin.

Dexpanthenol a skin vitamin - provides special protection and care for the skin.

### USE

After bathing, showering, swimming, and diving (freshwater or salt water) drop approx. 5 drops of NORMISON Ear Drops into the auditory canal, keeping your head tilted for 20-30 seconds to allow the ear drops to act.

Then bring the position of your head back to normal and wipe away the outflow. Let the fluid in the auditory canal evaporate. Repeat the above procedure for the other ear.

Do not clean your auditory canals mechanically and do not plug them with cotton.

NORMISON Ear Drops should be applied to the auditory canal at body temperature.

### IMPORTANT INFORMATION

NORMISON Ear Drops may cause a burning sensation in cases of slight irritation of the external auditory canal.

NORMISON Ear Drops should not be used if the auditory canal is inflamed or if the eardrum is perforated (hole in the eardrum).

NORMISON Ear Drops is not medicine. In case of earache seek medical advice.

Prior to NORMISON Ear Drops treatment cleaning of the external auditory canal with CERUSTOP Ear Oil is recommended if earwax, dead skin, or other deposits are present in the auditory canal.

The bottle should always be kept closed and out of the reach of children!

Do not store above 25 °C !

### WARNING

Do not insert any objects (e.g. cotton buds) into the auditory canal!

Do not swallow NORMISON Ear Drops or insert it into the eyes!

### PACKAGE SIZE

10 ml bottle with dropper pipette.

MANUFACTURER: MoNo chem-pharm Produkte GmbH, 1200 Vienna - Austria

DISTRIBUTION AUSTRIA: SIGMAPHARM Arzneimittel GmbH & Co KG, 1201 Vienna - Austria

DISTRIBUTION GERMANY: bene-Arzneimittel GmbH, 81479 Munich - Germany